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Project 12 SPRL
Fazantenstraat 14,
2811 Mechelen, Belgique

EUROPE - Community face coverings CWA 17553:2020

Comfort, Community or Barrier masks

Model "Breezy Mesh" - Type "Sport" - Reusable mask
Filtration efficiency to particles around 3 ($\pm 0,5$) μm : $\geq 90\%$ (Cat.1)

WARNINGS

This community face covering is not a medical device within the meaning of Directive 93/42/CEE or Regulation EU/2017/745 (surgical masks) nor is it personal protective equipment in the sense of Regulation EU/2016/425 (respiratory protective devices).

This community mask does not replace hygiene measures (such as washing hands regularly, social distancing, reducing contact with other people). It adds a physical barrier that allows the user to protect his environment from the projection of particles. It can also provide the user with limited protection against infectious agents.

To protect your health and that of others, it is very important to follow the user manual. If you are ill, this mask is not suitable, ask your doctor for advice. This mask is not intended for use by medical personnel.

CHECK BEFORE USE

Check the mask including all parts for damage before use. If the mask shows any kind of damage (lesser connection, tears, holes, deformation, shows of wear and tear, or other), the mask can no longer be used and must be replaced.

PUTTING ON BREEZY

- Step 1: Wash your hands with soap and water before touching it.
- Step 2: Check the mask for any damage, and replace the filter or valves if necessary.
- Step 3: Hold the mask with the adjustable nose bridge upwards, and the filter inwards.
- Step 4: Hold the two ear loops, place your chin in the mask, and gently pull them towards you. Hook them behind the ears so that you can attach the headband (if present). Position the mask so that it covers the nose, mouth and chin.
- Step 5: Adjust the nose bridge so that it takes on the shape of your nose. Do not touch the masks anymore from now on.

TIP: It is recommended to wear the mask on bare skin as having a beard can reduce filtration efficiency beyond the intended limits.

REMOVING THE BREEZY

- Step 1: Wash your hands with soap and water before touching it.
- Step 2: Do not touch the front of the mask as it can be infected.
Only touch the ear loops and the headband (if present).
- Step 3: Hold the two ear loops, loosen the headband, and gently lift them to remove the mask.
- Step 4: Store the mask in a closed container separate from each other, or in a closed laundry basket, and follow the maintenance procedure. See the "Maintenance recommendations" section.
- Step 5: Wash your hands with soap and water.

SIZES

Breezy is available in 3 sizes. Small (junior: 14cm H / 30cm L), medium (adult: 15,5cm H / 33,5cm L) and large (adult: 17cm H / 36cm L). To know the size, see the packaging.

TIP: Make sure not to tighten the mask too much so that it can follow your head movement smoothly. A good fit is required. Do not move the Breezy to eat, drink or talk.

WARNINGS

This product can only be used by children over 3 years old due to choking hazard. Children from 4 to 12 years must be supervised when wearing this product.

Wearing a mask requires some adjustment. You may have the feeling of having less breathing air. This is normal, the mask works. Please note, just like a normal mask, a sports mask does not allow heavy effort. Use Breezy continuously for up to 3 hours.

MAINTENANCE RECOMMENDATIONS

Breezy

Close the headband velcro (if present). Wash Breezy after each use with a neutral detergent without fabric softener, no dry cleaning, and separated with an added ballast (like an old bath towel) to generate a mechanical cleaning action. For disinfecting, a temperature of at least 60 °C (if possible - otherwise at a higher temperature) must be maintained for 30 minutes. Wash your hands after putting Breezy in the washing machine. Allow to dry before using again. Compliance guaranteed up to 10 washing cycles.

Filters & Valves

The filters and valves are replaceable. After each intensive use of the mask, let them dry until they are completely dry. The service life of the filters and the valves are closely related to the intensity of their use and the maintenance. Never use filters or valves that show signs of wear and tear or damage. Replace the filters after 1 month of use at the latest. You can not wash the filters or the valves!

FIBER COMPOSITION

80% polyester, 20% polypropylene

STORAGE CONDITIONS

Keep the container closed in a dry and well-ventilated place away from frost and direct light.

DISCLAIMER

Project 12 BV can not be held liable for the incorrect use and safety of the Breezy masks.

Manufactured in China
Imported by
Project 12 BV
Fazantensstraat 14,
2811 Mechelen, Belgium

EUROPA - Community face coverings CWA 17553:2020

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